

Instructions

Charging

The Happy Hat has a built-in battery that may be charged with the included USB > Type-C cable.

Charge the device for 2 hours.

The device has a battery life of up to 1 hour of continuous use (or 4 applications).

If the lights begin to dim, it's time to charge.

Specifications

Power Consumption	115W
Charging Port	Type-C
Wavelength	660nm, 850nm
LED Quantity	230pcs @ 0.5W
Net Weight	1.5 lb
Irradiance	120mW/ sq cm

Package Includes:

- 1 Happy Hat Poll Cap
- 1 USB-Type-C cable
- 1 User Manual



Happy Hat Poll Cap User Manual

Contact/Customer Support
northeastequiwellness.com
Message (201) 279-1875

Leave a review on our Facebook
Thank you for supporting a small business!



Application

- Place the poll cap on the head, taking care to gently push the ears through the holes.
- Wrap the velcro around the neck, behind the throat.
- Pull the velcro snugly over the forehead and secure in place, taking care to avoid the eyes.
- Wrap snugly so that the LEDs are pressed against the skin for maximum efficacy.



Usage

- Press the power button **once** to activate all wavelengths.
- Press the power button **twice** to activate red light (660nm) only.
- Press the power button **three times** to activate near infrared light (850nm) only. This setting will not produce visible light as NIR light is invisible to the human eye.
- Press the power button one more time until the green light turns off. The device has powered off.
- Hold the power button for 3 seconds to activate the pulsing mode. The button will turn blue, then you can cycle through the settings again.

Usage Guidelines

- Never use over a cancerous growth.
- Do not use on recent burns.
- Do not use over the thyroid if the horse has hyperthyroidism.
- Consult a veterinarian before use on lame, ill, pregnant, senior, or young horses.

