

Instructions

Charging

The bell boot has a built-in battery that may be charged with the included USB > Type-C cable.

Charge the device for 2 hours.

The device has a battery life of up to 1 hour of continuous use (or 4 applications).

If the lights begin to dim, it's time to charge.

Specifications

Power Consumption	40W
Charging Port	Type-C
Wavelength	660nm, 850nm
LED Quantity	80pcs @ 0.5W
Net Weight	0.33 lb
Irradiance	120mW/ sq cm

Package Includes:

- 1 Red-Light / Near-Infrared Bell Boot
- 1 USB-Type-C cable
- 1 User Manual



Brilliant Bell Boot User Manual

Contact/Customer Support
northeastequiwellness.com
Message (201) 279-1875

Leave a review on our website
or on Facebook!
It really helps us out!



Application

- Wrap the boot over the pastern/corony band area, as you would a regular bell boot.
- You may also use it over the knee, hock, or poll.
- Wrap snugly so that the LEDs are pressed against the skin for maximum efficacy.
- This boot is not meant to be used as an overreach boot or protective boot - it is a therapy device meant to be used in crossties or in the stall.



Usage

- Press the power button **once** to activate all wavelengths.
- Press the power button **twice** to activate red light (660nm) only.
- Press the power button **three times** to activate near infrared light (850nm) only. This setting will not produce visible light as NIR light is invisible to the human eye.
- Press the power button one more time until the green light turns off. The device has powered off.
- Hold the power button for 3 seconds to activate the pulsing mode. The button will turn blue, then you can cycle through the settings again.

Usage Guidelines

- Never use over a cancerous growth.
- Do not use on recent burns.
- Do not use on an open or bleeding wound.
- Consult a veterinarian before use on lame, ill, pregnant, senior, or young horses.

*Access your free "Cheat Sheet"!
A guide with usage
recommendations to support your
horse in any situation.*

